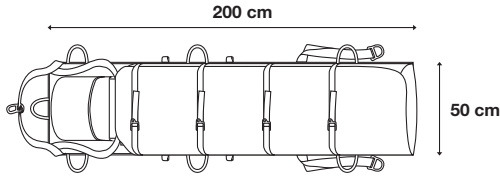
# NEST

**(EN) Litter for rescue in difficult access locations**

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12,8 kg

## WARNING

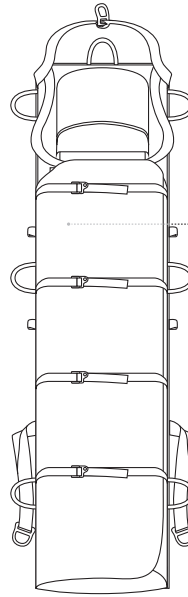
Activities involving the use of this equipment are inherently dangerous. You are responsible for your own actions and decisions.

- Before using this equipment, you must:
- Read and understand all instructions for Use.
  - Get specific training in its proper use.
  - Become acquainted with its capabilities and limitations.
  - Understand and accept the risks involved.

**FAILURE TO HEED ANY OF THESE WARNINGS MAY RESULT IN SEVERE INJURY OR DEATH.**

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**e. Individual number**  
00 000 AA 0000



**f.** Year of manufacture  
**g.** Day of manufacture  
**h.** Control or name of inspector  
**i.** Incrementation

PETZL  
ZI Cidex 105A  
38920 Crolles  
France  
**PETZL.COM**

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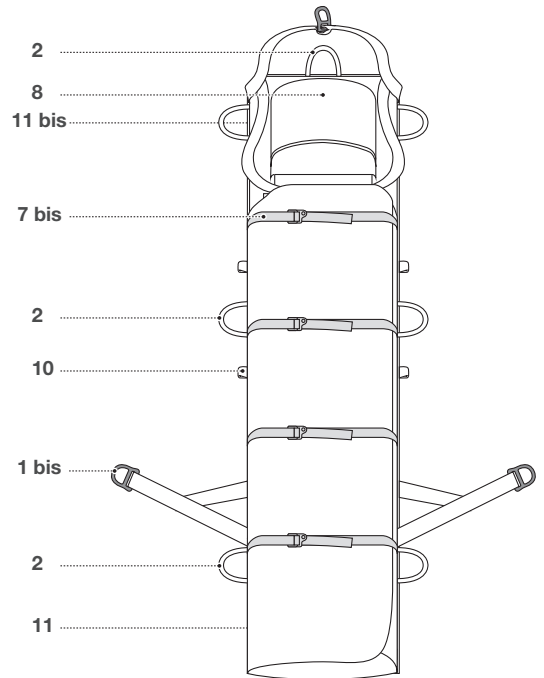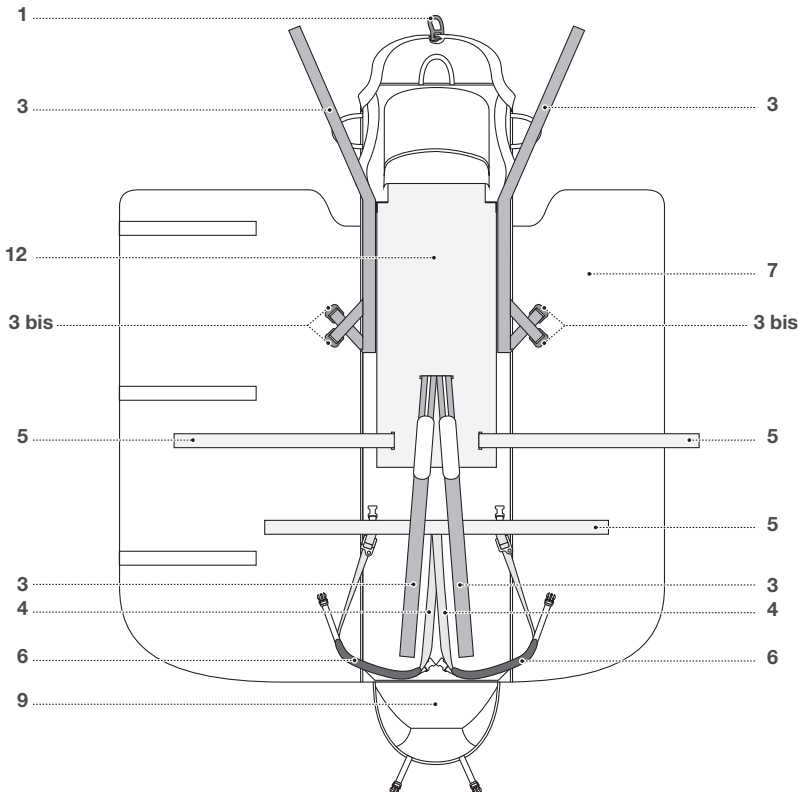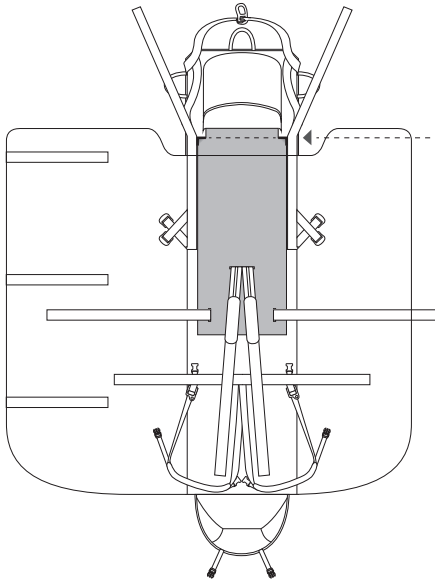
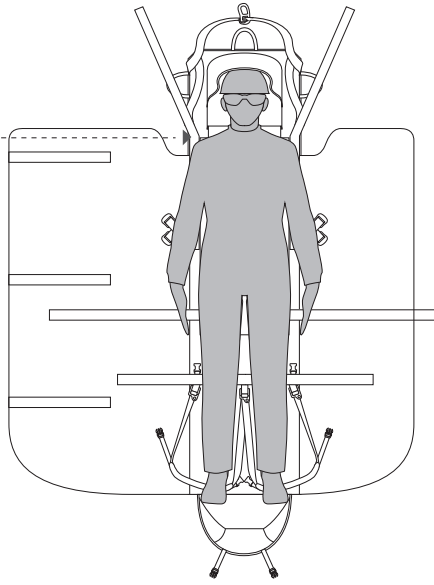
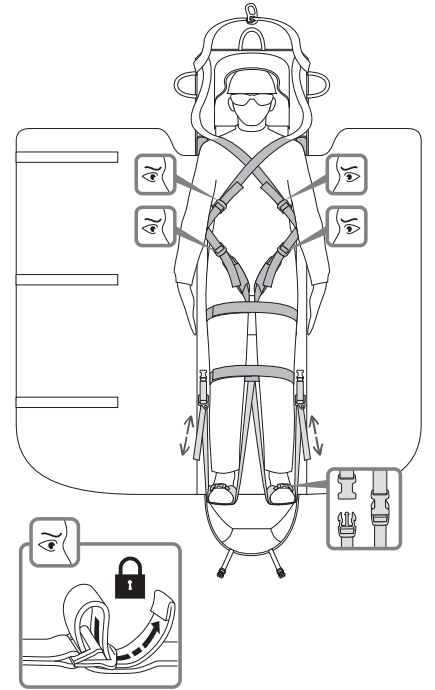
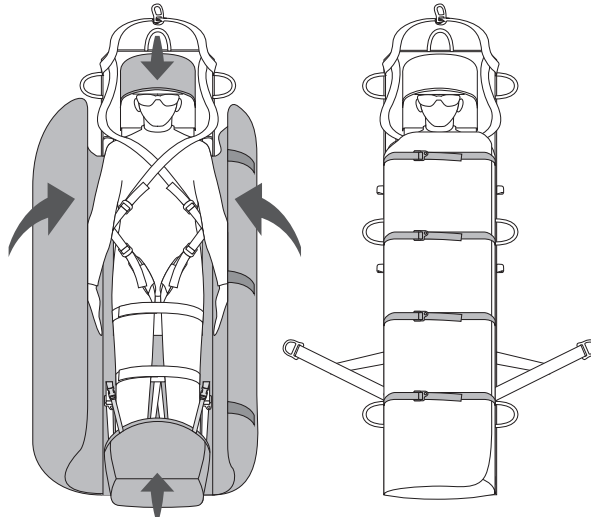
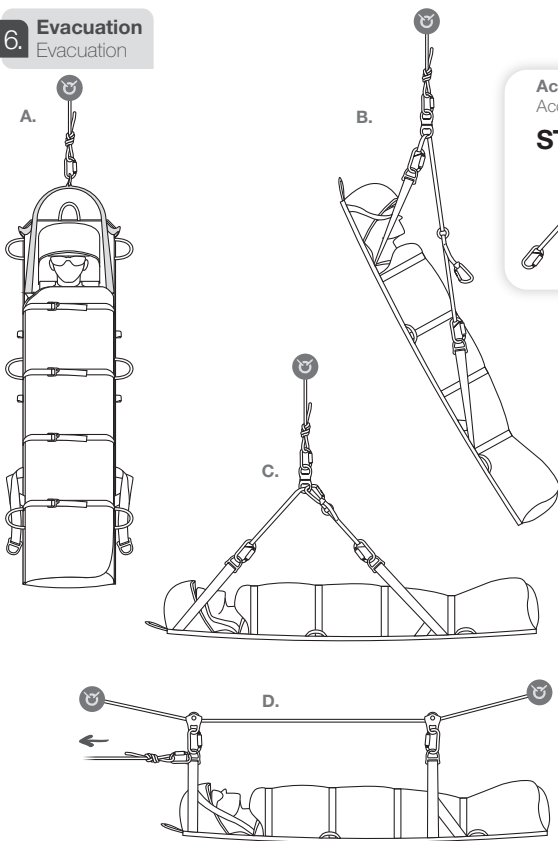
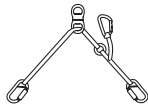
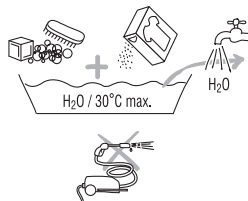
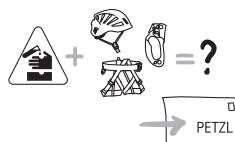
Only the techniques shown in the diagrams that are not crossed out, and that do not display a skull and crossbones symbol and/or «NO!» are authorized. Some examples of misuse are presented, but it is impossible to describe them all. Check [www.petzl.com](http://www.petzl.com) regularly to find the latest technical information. Contact Petzl if you have any doubt or difficulty understanding these documents.

## 1. Field of application

Litter (200x50x5 cm) for rescue in difficult access locations.

This product must not be loaded beyond its strength rating, nor be used for any purpose other than that for which it is designed.

### WARNING

**Activities involving the use of this equipment are inherently dangerous. You are responsible for your own actions and decisions.**

Before using this equipment, you must:

- Read and understand all instructions for use.
- Get specific training in its proper use.
- Become acquainted with its capabilities and limitations.
- Understand and accept the risks involved.

**Failure to heed any of these warnings may result in severe injury or death.**

### Responsibility

WARNING, specific training is essential before use. This training must be adapted to the practices defined in the Field of Application.

This product must only be used by competent and responsible persons, or those placed under the direct and visual control of a competent and responsible person. Gaining an adequate apprenticeship in appropriate techniques and methods of protection is your own responsibility.

You personally assume all risks and responsibilities for all damage, injury or death which may occur during or following incorrect use of our products in any manner whatsoever. If you are not able, or not in a position to assume this responsibility or to take this risk, do not use this equipment.

## 2. Nomenclature of parts

(1) Head end attachment point, (1 bis) Side attachment points, (2) Carry handle, (3) Body harness straps, (3 bis) Harness DoubleBack buckles, (4) Foot support adjustment straps, (5) Positioning straps, (6) Foot supports, (7) Flaps, (7 bis) Flap closure straps, (8) Head cover, (9) Foot cover, (10) Accessory attachment points, (11) Sleeve for longitudinal stays, (11 bis) Sleeve for transverse stays, (12) Foam pad.

## 3. Inspection, points to verify

### Before each use

Check the condition of the harness straps and attachment points.

Verify that the DoubleBack buckles operate properly.

Look for cuts in the webbing, wear and damage due to use, to heat, and to contact with chemical products, etc.. Be particularly careful to check for cut threads.

Consult the details of the inspection procedure to be carried out for each item of PPE on the Web at [www.petzl.com/pep](http://www.petzl.com/pep). Contact Petzl if you have any doubt.

### During each use

It is important to regularly monitor the condition of the product and its connections to the other equipment in the system. Make sure that all pieces of equipment in the system are correctly positioned with respect to each other.

## 4. Compatibility

Verify that this product is compatible with the other elements of the system in your application (compatible = good functional interaction).

## 5. Installing the patient

The litter is designed for optimal comfort for a patient weighing 120 kg maximum and height between 1.50 m and 1.90 m.

- a. Place the litter on a stable base. Open the flaps and lay them aside. Take the body harness straps out of the buckles. Loosen the foot support straps.
- b. Place the patient on the litter. The top of the foam pad corresponds to the placement of the patient's shoulders, regardless of his height. The patient must be wearing a helmet and protective eyewear.
- c. Buckle the harness straps. Fasten the positioning straps. Adjust the length of the foot support straps and fasten the foot support buckles. Make sure all of the straps are tight enough to hold the patient securely, without creating painful pressure points.
- d. Fold the head and foot covers over and fasten the head and foot cover buckles. Close the flaps, or fold them to the sides, depending on the temperature, water level, etc... Tighten the flap straps The patient's arms can be inside or outside of the litter.

Some safety elements may or may not be secured in place, depending upon the nature of the patient's wounds as determined by the physician, and the difficulty of the evacuation.

## 6. Evacuation

For reasons of clarity, the drawings do not show all of the techniques used during rescue with a litter. Get training in these techniques before using the litter.

WARNING: do not haul the litter from the carry handles.

### A- Vertical litter

Evacuation is effected using the head end attachment point. Adapted for narrow openings. Limits rockfall risk for the patient Minimum patient comfort.

### B- Near-vertical litter

We advise you to use the STEF (S59). Good comfort/bulk compromise.

### C- Horizontal litter

We advise you to use the STEF (S59). Comfortable for the patient, but increases bulk and patient exposure to rockfall. Can be ordered by the doctor.

### D- Tyrolean traverse

## 7. Transporting the litter

The litter can be stored in the carrying case (S62). To transport the litter in a vehicle, you can remove the longitudinal stays and fold it.

## 8. Petzl general information

### Lifetime / When to retire your equipment

For Petzl's plastic and textile products, the maximum lifetime is 10 years from the date of manufacture. It is indefinite for metallic products.

WARNING: an exceptional event can lead you to retire a product after only one use, depending on the type and intensity of usage and the environment of usage (harsh environments, marine environment, sharp edges, extreme temperatures, chemical products, etc.).

A product must be retired when:

- It is over 10 years old and made of plastic or textiles.
- It has been subjected to a major fall (or load).
- It fails to pass inspection. You have any doubt as to its reliability.
- You do not know its full usage history.
- When it becomes obsolete due to changes in legislation, standards, technique or incompatibility with other equipment, etc...

**Destroy retired equipment to prevent further use.**

### Product inspection

In addition to the inspection before each use, an in-depth inspection must be carried out by a competent inspector. The frequency of the in-depth inspection must be governed by applicable legislation, and the type and the intensity of use. Petzl recommends an inspection at least once every 12 months.

To help maintain product traceability, do not remove any markings or labels. Inspection results should be recorded on a form with the following details: type of equipment, model, manufacturer contact information, serial number or individual number, date of manufacture, date of purchase, date of first use, date of next periodic inspection, problems, comments, the name and signature of the inspector. See an example at [www.petzl.com/pep](http://www.petzl.com/pep).

### Modifications, repairs

Modifications and repairs outside of Petzl facilities are prohibited (except replacement parts).

### Storage

Store the product in a dry place away from exposure to UV, chemicals, extreme temperatures, etc... Clean and dry the product if necessary.

### 3-year guarantee

Against all material or manufacturing defect. Exclusions: normal wear and tear, oxidation or alterations, incorrect storage, poor maintenance, negligence, uses for which this product is not designed.

### Responsibility

Petzl is not responsible for the consequences, direct, indirect or accidental, or any other type of damage befalling or resulting from the use of its products.

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© Petzl